

170308



LifeStyles® SKYN® Brand Condoms deliver protection against pregnancy and, if used properly, will help reduce the risk of transmission of HIV (AIDS) infection and many other sexually transmitted infections.

No. 3003



THE NEXT GENERATION OF CONDOMS

This isn't just a box of condoms. This is the latest in condom technology for a truly intimate sexual experience. Our SKYNFEEL™ material is soft and comfortable, for the most natural fit and feel. You are holding a ticket to an experience of incredible sensitivity, that allows you and your partner to feel everything. **This isn't just a box of condoms. This is the closest thing to wearing nothing.**

ELITE



- STRAIGHT SHAPE
- LONG-LASTING ULTRA-SMOOTH LUBRICANT
- ULTRA-THIN AND ULTRA-SOFT FOR AN EXCEPTIONALLY SENSITIVE FEELING. SKYN® ELITE CONDOMS TAKE THE EXPERIENCE TO THE NEXT LEVEL.**
- STRENGTH OF PREMIUM LATEX
- NATURAL COLOR

IMPORTANT CONSUMER INFORMATION

LIFESTYLES® SKYN® SYNTHETIC POLYISOPRENE CONDOMS DO NOT CONTAIN NATURAL RUBBER LATEX AND ARE A SUITABLE CHOICE FOR PEOPLE WITH A KNOWN OR SUSPECTED ALLERGY TO NATURAL RUBBER LATEX. FOR MAXIMUM PROTECTION, READ AND FOLLOW THE INSTRUCTIONS AND CAUTION ADVICE PRINTED INSIDE THIS CARTON.

www.skyn.com



#1 NON-LATEX CONDOM

THINNER & softer



Made from SKYNFEEL™, a technologically advanced non-latex material, which feels so soft and comfortable, that you'll find it barely noticeable


ELITE

Ultra Thin & Ultra Soft for a Comfortable Fit

12
NON-LATEX LUBRICATED CONDOMS



Each condom is electronically tested to meet highest standards of safety and reliability

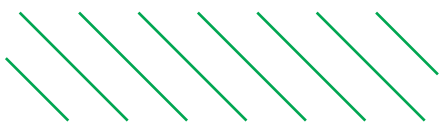

AVOID EXCESS HEAT. STORE AT ROOM TEMPERATURE.

© and ™ are trademarks owned by LifeStyles Healthcare Pte Ltd. ©2019 LifeStyles Healthcare Pte Ltd
Patents: www.lifestyles.com/our-patents

Mfd. for/Fab. Pour: Suretex Limited, 31/1 Moo4, Suratthani-Thakuapha Road
Tambon Khao Hua Kwai, Amphur Phunphin Suratthani 84130, Thailand



CAUTION: SKYN® polyisoprene condoms do not contain natural latex and are a suitable choice for people who have a known or suspected allergy to natural rubber latex. Some highly sensitive individuals may, however, experience allergic reactions to ingredients in these condoms. Should you experience a reaction, irritation or burning sensation, discontinue use and seek medical advice.

Effectiveness

Although no contraceptive can guarantee 100% effectiveness, condoms, when properly used, are a highly effective means of preventing pregnancy without the serious side effects sometimes associated with some other methods. Condoms are intended for use in vaginal intercourse. Condoms, when properly used, can also aid in reducing the risk of spreading HIV infection (AIDS) and many other sexually transmitted infections (STIs).




Directions for Use

Condoms are intended to prevent pregnancy, HIV/AIDS, and other sexually transmitted infections.

1. Use a new condom for every act of sex.
2. Tear open the package carefully. Do not use fingernails, teeth or anything that can damage the condom.
3. Before any sexual contact, place the condom on the head of the erect penis with the rolled side out.
 - Squeeze tip of condom to remove any air. (Excess air could cause the condom to break).
 - Lesions, pre-ejaculate secretions, semen, vaginal secretions and blood can all transmit infectious organisms.
4. Unroll or pull the condom all the way to the base of the erect penis. If the condom doesn't unroll easily, it may be on backwards, damaged or too old. Throw it away and start over with a new condom.
5. Immediately after ejaculation, hold the rim of the condom in place and withdraw the penis while it is still erect.
 - Avoid spilling semen.
6. Dispose of a used condom by wrapping it in tissue and throwing it into the trash. Wash your hands with soap and water.

Precautions:

- Do not reuse condoms.
- Store condoms in a cool (below 100°F), dry place and avoid exposure to direct sunlight. Protect condoms against prolonged exposure to extreme low or high temperatures, moisture and fluorescent light.
- If the rubber material is sticky or brittle or obviously damaged, do not use the condom.

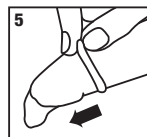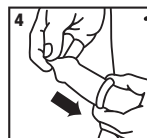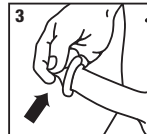

- If the color is uneven or changed, do not use the condom.
- If additional lubricant is desired, use a LifeStyles® water-based lubricant.
- DO NOT USE OIL-BASED LUBRICANTS, such as those made with petroleum jelly, (e.g., Vaseline®), mineral oil, vegetable oil or cold cream as these may damage the condom.

Degree of STI Protection:

Condoms reduce the risk of transmitting STIs by providing a barrier against the source of the infection.

- Condoms are most effective against STIs such as HIV infection (AIDS) and gonorrhea that are spread by contact with the head of the penis.
- Condoms are less effective against STIs such as Human Papillomavirus (HPV) and herpes. These STIs can also be spread by contact with infected skin that is not covered by the condom.

If you believe you have an STI, contact a health care provider. For more information on condoms or STIs, contact a health care provider or a public health agency.

ALL SKYN-USA-PI-USA-REV1